




# GreenEgg™ Flourescent Vaginal & Rectal Manipulator

With Tissue Reveal Technology™

## Instructions for Use

### DEVICE DESCRIPTION

The GreenEgg™ is a sterile, single use handheld, non-powered pelvic manipulator used for surgical manipulation and transillumination of the rectum and/or vagina. The 30mm diameter polymer tip fluoresces when viewed under a near infrared system.

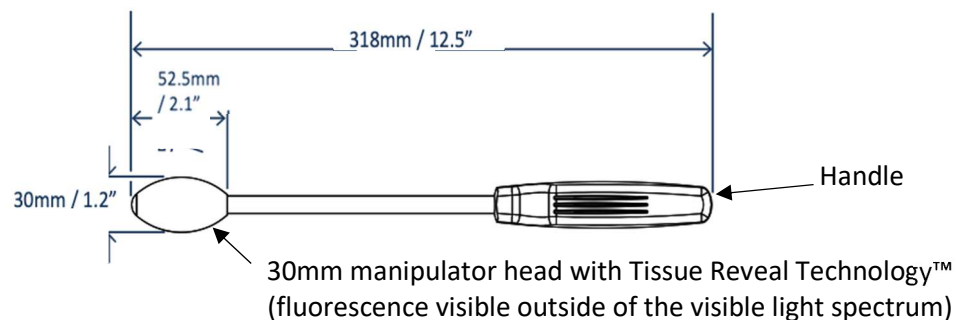

### PACKAGING CONTENTS

A single GreenEgg™ device that is CSR wrapped and placed inside a Tyvek header bag in sterile fashion.

### INDICATIONS FOR USE

The EndoGlow GreenEgg™ manipulator is indicated to provide manipulation of the vaginal or rectum during minimally invasive surgical procedures. When viewed with the NIR camera system of a laparoscope or surgical robot, the GreenEgg™ will fluoresce providing transillumination.

### DIRECTIONS FOR USE

#### Opening

1. Confirm expiration date.
2. Remove bag from box.
3. Open the bag using sterile technique.
4. Transfer the contents of the bag (CSR blue wrapped GreenEgg™ manipulator) to sterile field.

#### Preparation

1. Once introduced to the field, the CSR wrap can be removed from the device and discarded.
2. The GreenEgg™ manipulator is ready for use.

#### Insertion

- Insert the GreenEgg™ manipulator into the vagina or rectum and proceed with surgical procedure.

### WARNINGS

- Contents supplied sterile. Do not use the GreenEgg™ if the sterile barrier is damaged.
- Do not use the GreenEgg™ if the device has been opened or damaged.

## REUSE PRECAUTION STATEMENT

- For single use only: Do not reuse, re-process or re-sterilize.
- Attempting to re-sterilize and reuse the product will diminish its intended performance in transilluminating tissue, reducing its effectiveness. To ensure optimal results and patient safety, a new product must be used for each procedure.

## CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

## STORAGE

Store at room temperature, avoid extreme temperatures.

## DISPOSAL

Dispose of in accordance with all applicable Federal, State, and local Medical/ Hazardous waste practices.

## EXPLANATION OF SYMBOLS

 REF	Catalog number		Do not re-use
	Use by date		This product must not be re-sterilized
	Consult instructions for use		Do not use if package is damaged
	Sterilized using ethylene oxide		Product does not contain latex
	Keep away from sunlight		Manufacturer

**Rx only** U.S. Federal law restricts this device to sale by or on the order of a physician

## TRADEMARKS

- Tissue Reveal Technology™ is a trademark of EndoGlow, LLC.
- The GreenEgg™ is a registered trademark of EndoGlow, LLC.

Made in the USA



40 Franklin St.  
Rochester, New York 14604  
Phone: (315) 273-0005  
Email: [support@endoglow.com](mailto:support@endoglow.com)



[www.endoglow.com](http://www.endoglow.com)